

Fabric specs:

100% cotton fabric, tightly woven, minimum of 160 thread count per square inch.

Ties and tape – 100% cotton – white or natural color – minimum ¼" thick

Nose piece – minimum 4" long – plastic coated wire twist ties

Mask size when complete – 7" wide by 3.75"

3 pleats per mask facing upward (from outside view)




Additional Technical Specifications:

<https://www.modehausbridal.com/modehausmaskprintablepattern>


Surgical Mask

3.25.20

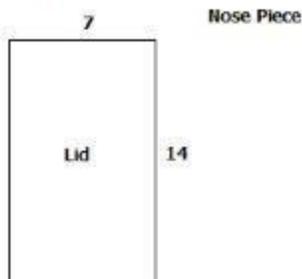

Serger


Fold the lid in half length wise, serge around 3 sides, stopping .5" from one corner.

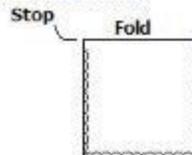

Sewing Mach.


Sew a .25 seam across the fold at the top.

Using template, mark 6 notches on each side.

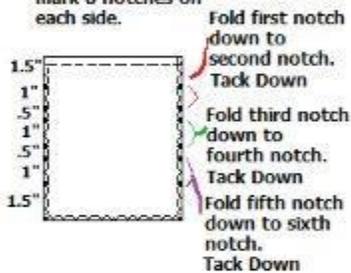

Do other side the same, make sure folds are going down on both sides.

Folder

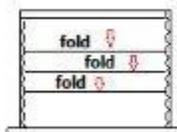

Take to folder, sew binding across the bottom. Snip even with the edges.

Sew binding about 20" long, insert one side of the mask into the binding, sew across and sew another 20" and cut.

Put nose piece in the opening that you left when serging. Sew the binding the same as other side.

